

PREVENTIS *SmarTest*[®]

FIT Home

Take your colorectal
cancer screening in
your own hands



Rapid test & app: The innovative colorectal cancer screening



Based on decades of experience in colorectal cancer screening, we have developed the unique test system Preventis SmarTest® FIT Home for the early detection of colorectal cancer.

With this innovative test system even smallest amounts of blood can be detected in stool and the concentration contained can be determined precisely.

Preventis SmarTest® FIT Home is the combination of an immunological rapid test and the SmarTest FIT® app for quantitative determination of blood in stool samples.


Take your colorectal cancer screening into your own hands

Early diagnosed colorectal cancer is almost always curable.

The examination of stool for occult (hidden) blood offers a great opportunity to detect colorectal cancer at an early stage without invasive procedures such as colonoscopy. Even in the early stages - long before the onset of symptoms - tumors can cause bleeding, where the blood is excreted in the stool. This immunological stool test detects even the smallest amounts of blood that are not visible to the naked eye.

From the age of 50, every insured person in Germany is entitled to regular examinations for the early detection of colorectal cancer.


How does colorectal cancer screening work?

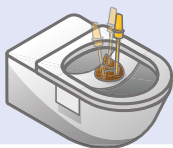

Simple and hygienic
stool sampling



Sample application to
the test cassette

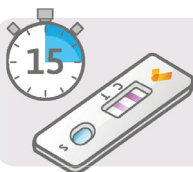

15 minutes
incubation time



Evaluation by the
SmarTest FIT® app



The test result is
displayed **immediately**
on the smartphone



In case of a **negative** test
result, **annual testing**
is recommended as a
precautionary measure

What are the advantages of the rapid test?

- ✓ **Preventis SmarTest® FIT Home** detects **smallest amounts of blood** in stool, determines the **concentration** and is highly reliable
- ✓ There is **no need for a diet** prior to the test
- ✓ **Evaluation at home** eliminates the need to transport the stool sample to the practice and the long uncertain wait for your test result
- ✓ The test result is already available after **15 minutes**

A positive result must lead to further medical clarification. Blood in stool can also have other, sometimes harmless, causes, such as inflammation or hemorrhoids. **If your test result with Preventis SmarTest® FIT Home is positive, consult a medical professional and discuss the further procedure.**

SmarTest FIT® app
download here for free





WWW.PREVENTIS.COM

Distributed by:

Preventis GmbH
Stubenwald-Allee 8a
64625 Bensheim, Germany
T: +49 6251 70711-0
F: +49 6251 70711-299
INFO@PREVENTIS.COM
WWW.PREVENTIS.COM



PREVENTIS
QuantOn®
Technology